



Export to Excel Spreadsheet... Four Easy Steps

1

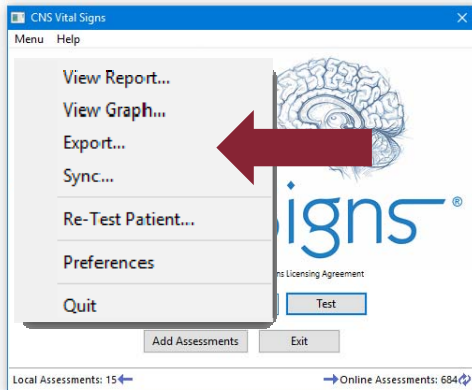

Open the CNS Vital Signs Application... CLICK MENU ...CLICK 'EXPORT'


2

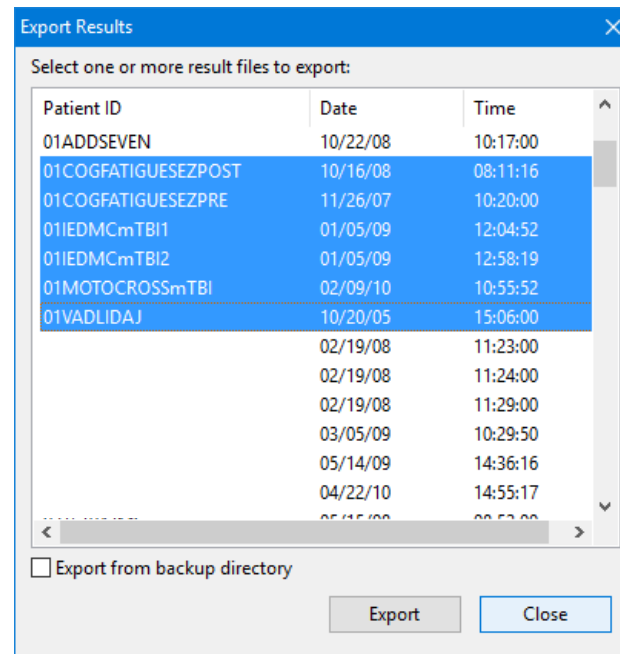

Next, highlight the test or tests for export. Click the Export button at the bottom of the window.

Create an EXPORT FILE... Save assessment data to the export file folder... CLICK OK

3

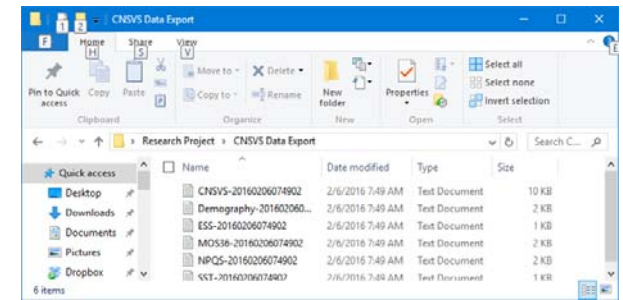

The export file is saved in a TAB DELIMITED Notepad file. DRAG the Notepad File into the Spreadsheet and the assessment data will auto-populate.

4

	A	B	C	D	E
1	test_date	test_time	subject_id	birth_date	signat.
2	1/5/2010	9:51:49	01121943EL	1/12/1943	5d16-49z
3	10/16/2010	8:11:16	CogFatigueADC	8/19/1952	5d16-4923
4	12/15/2010	12:48:20	CogFatigueADC	8/19/1952	5d16-45
5	3/23/2011	10:48:38	CollegeADDpre	7/30/1989	e5ae-5c3a
6	4/22/2011	11:01:36	CollegeADDpos	7/30/1989	e5ae-5c3a
7	10/16/2010	9:01:21	cogfatigueadd	8/18/1954	e5ae-5
8	3/23/2011	9:50:36	CollegeADDpre	7/30/1989	e5ae-5c3
9	9/9/2011	15:49:20	npq	2/2/1901	3862-7e77
10	9/9/2011	13:42:58	JDD122353	12/23/1953	3862-7e77
11	11/4/2011	23:06:34	zung	4/6/1952	3862-7e7

Data EXPORT Format... ROWS: Patient data for each unique assessment testing session, and COLUMNS: Testing Data for each variable with the variable name described in the header row.

How do I export test data for analysis?

