

ReTest a Patient

ReTest a Patient:

After testing a patient...

- A. Return to the Test Administrator Screen
- B. SELECT the 'ReTest' button to begin the re-test e.g., invalid effort, misunderstood instructions, did they need their glasses, etc.
- C. SELECT the patient to re-test
- D. SELECT the 'ReTest' button

The re-test will begin.

This feature allows you to re-test any patient, using the same Patient ID, who has taken CNS Vital Signs **within a 24 hour period of time** without being charged an additional session. This feature is especially helpful for those subjects who may have misinterpreted the directions during the initial testing or when doing a test-dose paradigm.

This is a KEY ADVANTAGE as CNS Vital Signs was designed for serial administration. The words and symbols in the tests are auto-randomized from a large bank of potential stimuli generating an unlimited number of alternate forms. This minimizes potential learning effects during serial administration. Tests other than memory tests are routinely administered several times a day in research settings.

CNSVS Online Assessment - Google Chrome

https://

CNS Vital Signs®

A Test Administrator: **cogtester**

Patient ID:

Remaining Assessments: 680

B

[Change My Password](#)

CNSVS Online Assessment - Re-Test Patient - Google Chrome

https://

CNS Vital Signs®

D

C	Patient ID	Test Time (GMT)
	patienttest	2016-02-07 13:47:10

How do I re-test a patient?