

Quick Start Guide


NEW ONLINE USERS: Access to the CNSVS Online testing app can only be activated by the CNS Vital Signs support staff. Email or call our support staff and let them know you would like to use CNSVOnline.com and they will be able to activate your web testing.

Go to CNS Vital Signs Online www.CNSVSONLINE.com; CLICK the 'Laptop Computer' Button; see below...




After a few seconds a LOG-IN Box will appear... enter the assigned Test Administrator and Password Codes

ENTER:
Subject or Patient ID and **PRESS TEST**

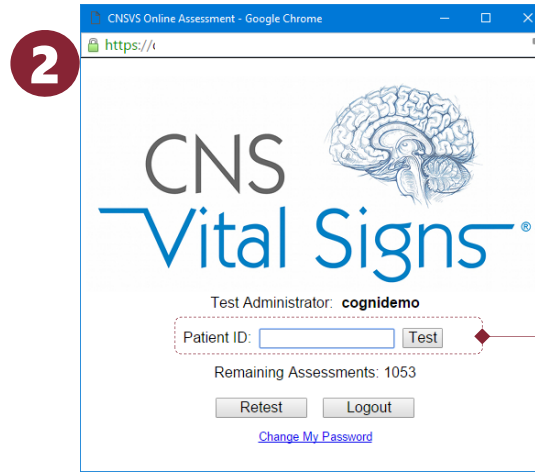


Important: CREATE a PRACTICE POLICY for Identifying Patients. All test results are longitudinally matched based on the unique Patient ID for each patient or research subject.


Demographics (Date of birth must be completed to calculate the age matched norms) **PRESS OK**

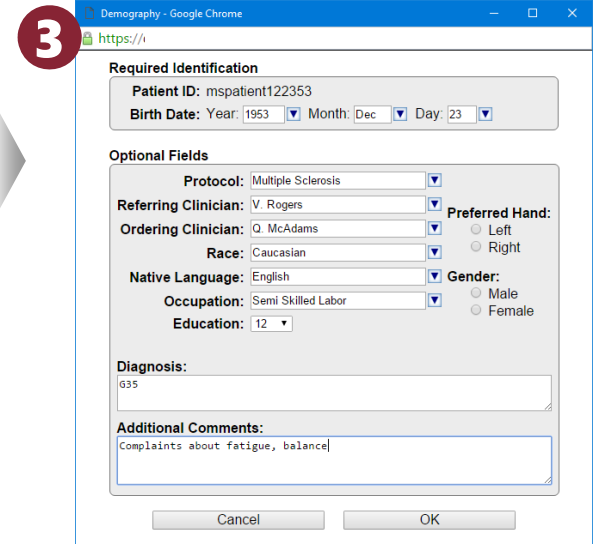

Confirm Language, Test and Scale Selection **PRESS OK**

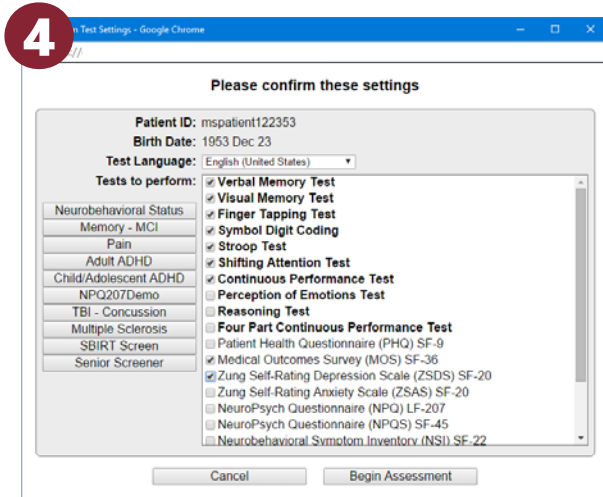

Testing is Available in 60 Languages.

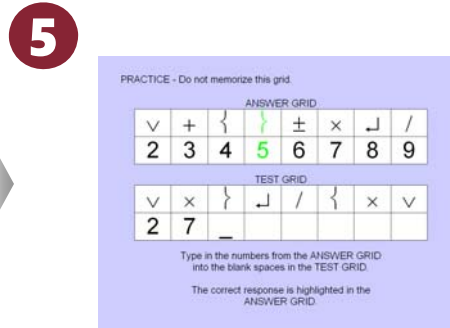

When Test is Complete Simply Access the AUTOMATICALLY SCORED REPORT (may require PW or can be configured to deny the viewing of the report except from the Admin Account)

Patient / Subject Takes the Tests (about 25 to 30 minutes)



...the AUTOMATICALLY SCORED TESTING REPORT(S) is generated for Interpretation and Integration into a final report.