

Accessing and Viewing CNSVS Online Testing Results



To Access your testing results and View Reports GO TO: www.cnsvsonline.com

CLICK the 'View Reports' button at the top left hand corner and bottom of the CNSVSONline.com webpage or go to CNSVS.com and select the 'Sign In - Manage Account' button... after a few seconds enter the **Username** and **Password** assigned to you. Rating Scales can be administered and reports viewed on browser enabled tablets.

After LOGIN the ACCOUNT FOLDER will be presented. The account folder will allow Users to VIEW REPORTS, GRAPH a Longitudinal View, view an Audit Log of account activity, EDIT Account Data, and Place a Purchase.

VIEW REPORTS: Clients can view reports by clicking the (A) 'View Reports' button. The correct Patient ID and report can be highlighted and SELECTED by clicking the VIEW REPORT button. **The selected report will open in a PDF format.**

CNSVS Online has an intuitive (B) report query toolset that will help the clinic rapidly access a patients testing results.

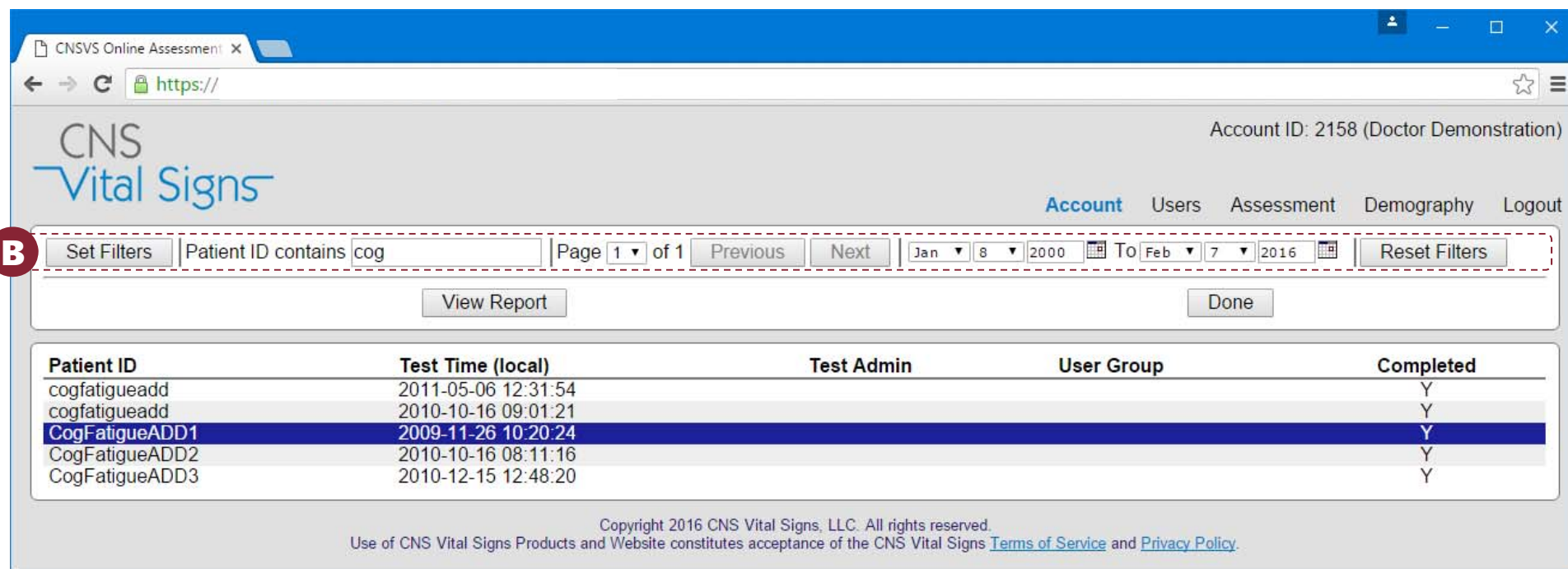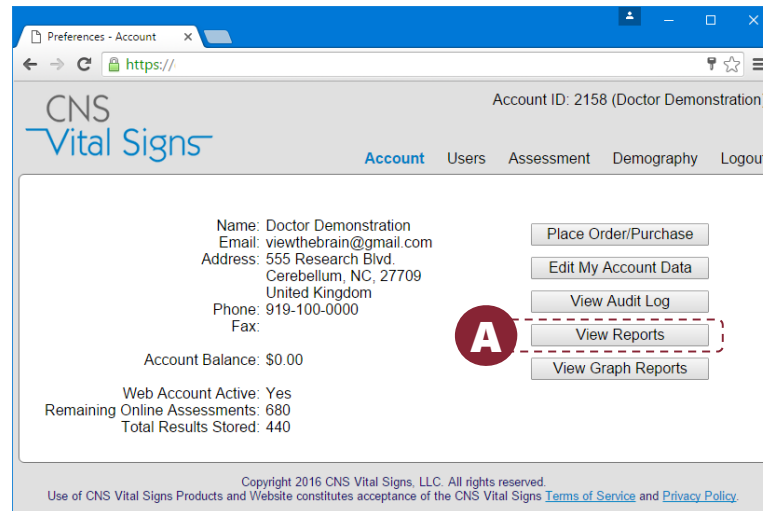

How do I access my testing results?

