


How To Customize and Configure Toolboxes & Test Panels


1

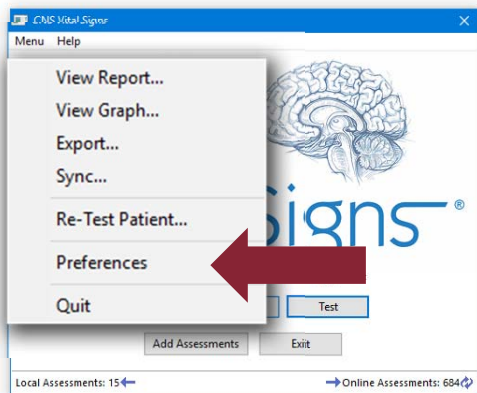

Open the CNS Vital Signs Application...
CLICK MENU... CLICK 'PREFERENCES'.
ENTER & OK 'cnsvs' (lower case)

CLICK the 'Assessment' tab to see the
current Toolbox configuration.


2

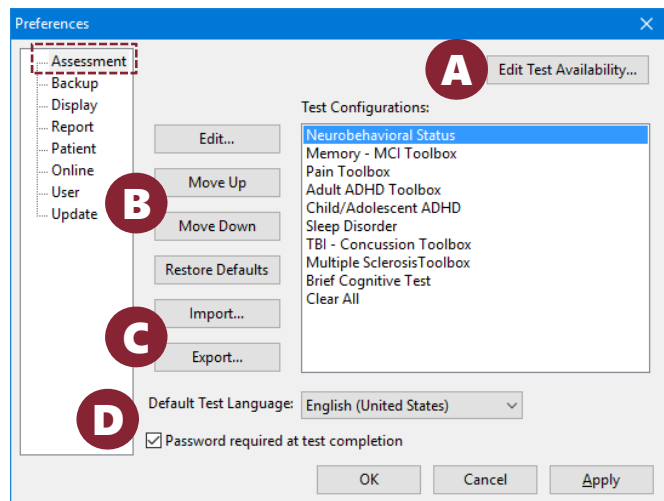

The 'Assessment' window allows you to...

- A. View **all the tests and scales** by clicking the **'Edit Test Availability'** button. Enable or activate tests and scales by checking the test(s) or scale(s)
- B. Arrange the toolbox order
- C. Export (MAKE) or Import 'config files' that will automate the configuration process
- D. Select a Default Test Language, and disable the PW requirement at the end of the testing session.


3

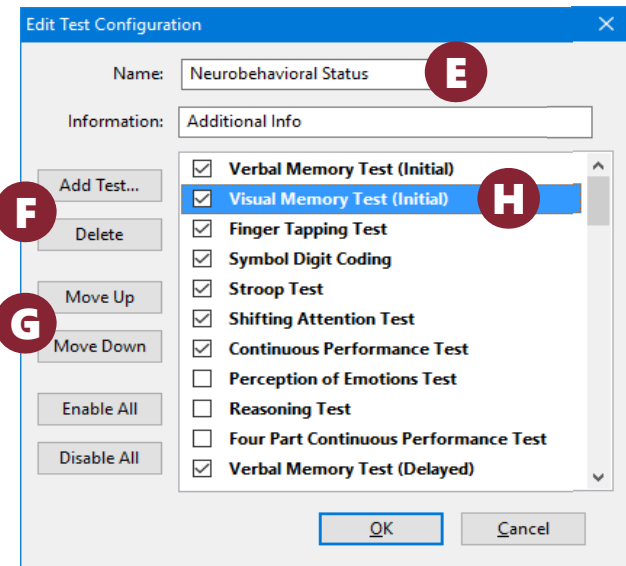

The 'Edit Test Configuration' allows you to...

- E. EDIT the Toolbox Name
- F. EDIT Test and Scale availability
- G. Arrange the Test and Scale order
- H. Select a standardized testing scenario.

CLICK OK... then CLICK the 'Apply' button to
make the changes.

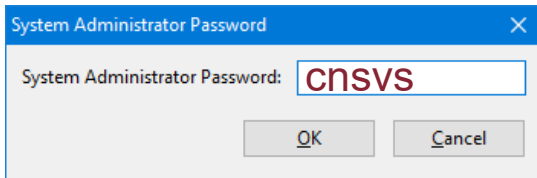

How do I configure testing panels?