


How To Sync Local Software Test Reports to your CNS VS Account

CNS VS local software users will need to turn-on your account report storage capability by... First, contacting CNS VS Support 888.750.6941 and ask for the service to be turned on... Second, follow the instructions below.

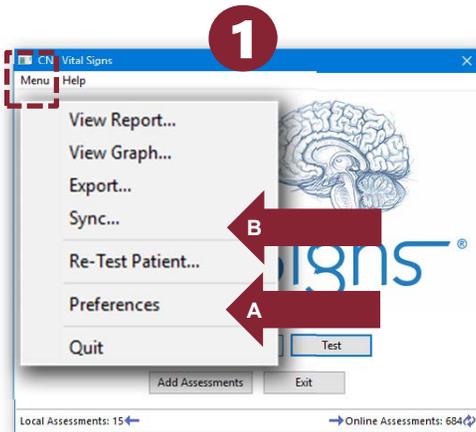


To Set-up Sync

Open the CNS Vital Signs Application... CLICK MENU...

- A. CLICK PREFERENCES to configure the SYNC function... enter 'cnsvs' (lower case)



Multiple testing computers in multiple locations e.g., telemedicine, ACO, etc. can also have their results backed-up to CNSVS Online. Testing results can be remotely accessed by clinicians with user permissions.

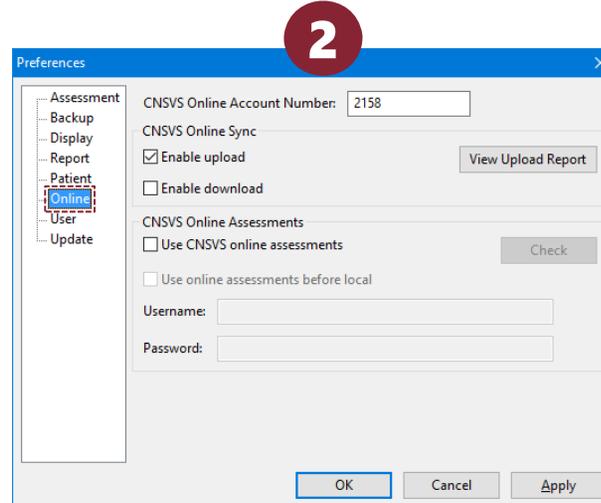


Configure SYNC Functionality

CLICK the ONLINE tab

- Enter your account number.
- CHECK the 'Enable upload' box to back-up testing data.
- CHECK the 'Enable download' to replicate the testing data or download remote testing results e.g., telemedicine.

Click Apply; Click Ok; Exit 'Preferences'

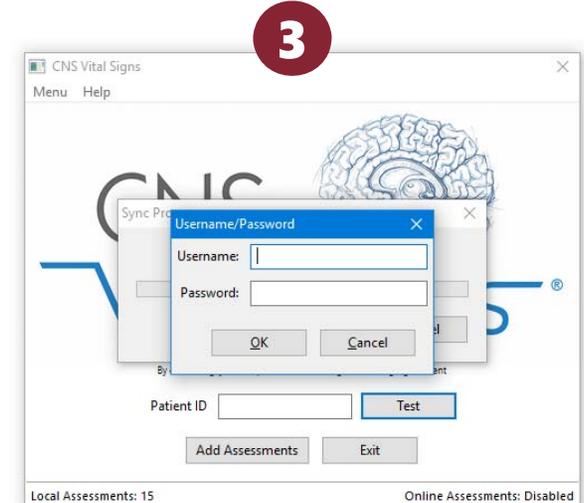

To Launch a Data Sync

- B. Open the CNS Vital Signs Application... CLICK MENU ...CLICK SYNC to upload and/or download patient reports

Activate a Data Sync

Insert Username and Password to SYNC your LOCAL Software Testing Data to your CNS Vital Signs Account

NOTE: To SYNC Your Data to a CNS Vital Signs Web Account you must FIRST have the CNSVS Support Team activate your CNSVS Online capability.

How do I SYNC/Store my testing data?

