

# Performing A CNS VS Web Assessment

## Steps to Perform A Web Assessment

- 1 Click on the 'CNSVS Online' button in the bottom right corner of the CNS Vital Signs homepage. ([www.cnsvs.com](http://www.cnsvs.com))
- 2 Select the 'Computer' icon to perform a neurocognitive assessment.
- 3 Enter your username and password and press 'Login'.
- 4 Enter a 'Patient ID' that will be used as a unique identifier for that patient and press 'Test'.
- 5 Enter the patient's birth date using the dropdown menus and press 'OK' when done.
- 6 Choose the appropriate assessment configuration settings for the patient and press 'Begin Assessment' when the patient is ready to start their testing session.

**Note: If you are unfamiliar with our test administration, please see the attached administration checklist.**

**1** View a Sample Report, Forgot My Password, NEW Telemedicine Solution Video, FAQs, **CNSVS Online**

**2** **COMPUTER** (Neurocognitive Tests and Rating Scales, Access Account and View Reports), **TABLET** (Rating Scales Only, Access Account and View Reports)

**3** User Name: ericstestaccount, Password: [dots], **Login**

**4** Test Administrator: ericstestaccount, Patient ID: 0002CNSEE3662, **Test**, Remaining Assessments: 38

**5** Required Identification, Patient ID: 0002CNSEE3662, Birth Date: Year: 1955, Month: Jan, Day: 06, **OK**

**6** Patient ID: 0002CNSEE3662, Birth Date: 1955 Jan 06, Test Language: English (United States), Tests to perform: [checkboxes for Verbal Memory Test, Visual Memory Test, Finger Tapping Test, Symbol Digit Coding, Stroop Test, Shifting Attention Test, Continuous Performance Test, Perception of Emotions Test, Reasoning Test, Four Part Continuous Performance Test, Adult ADHD Self-Report Scale (ASRS-v1.1) Symptom Checklist LF-18], **Begin Assessment**

# CNS Vital Signs Test Administration Checklist



## READ ALL INSTRUCTIONS

Make sure you understand the instructions before continuing.



## MINIMIZE POTENTIAL DISTRACTIONS

Turn off or silence all electronics, and make sure there are no surprises during the assessment.



## COMPLETE TEST IN ENTIRETY

Once you begin, you cannot pause, go back, or restart, so get comfortable before starting the assessment.



## DO PRACTICE WHEN AVAILABLE

Some sections will have practice prior the scored section. Completing them will help solidify instructions.



## ACCURACY FIRST, THEN SPEED

Correctly respond to the best of your ability, while pushing yourself to perform quickly.