

Performing A CNS VS Web Re-Test

About the Re-Test Function

Occasionally a patient will experience an issue during their assessment (quitting, computer failure etc.) or will have a results that triggered an invalid score in their report. The Re-Test function provides users with the ability to create and customize a new testing session for a patient without using another assessment credit. Often, users will create Re-Tests for only one or a few normed cognitive subtests. Patients are eligible for retesting within 1-week of their initial assessment.

If you need a merged report after a Re-Test is complete, please send an email to support@cnsvs.com with the subject 'Merge Request' explaining why the merge is needed, along with PDF copies of the reports.

Steps to Perform A Web Re-Test

- 1 Click on the 'CNSVS Online' button in the bottom right corner of the CNS Vital Signs homepage. (www.cnsvs.com)
- 2 Select the 'Computer' icon to perform a neurocognitive assessment.
- 3 Enter your username and password and press 'Login'.
- 4 Select 'Retest' to see a list of patients eligible for retesting. Note: Patients IDs are eligible for retesting for seven days following their most recent test.
- 5 Select one of the Patient IDs from the list and then press the 'Retest' button.
- 6 On the next screen, use the dropdown menus to select the 'Birth Date' for that patient.
- 7 Customize the assessment configuration settings for the patient's Re-Test and press 'Begin Assessment' when the patient is ready to start their new testing session.

1 View a Sample Report | FAQs | **CNSVS Online** | Forgot My Password | NEW Telemedicine Solution Video

2 **COMPUTER** Neurocognitive Tests and Rating Scales | **TABLET** Rating Scales Only

3 User Name: ericstestaccount | Password: [dots] | **Login**

4 Test Administrator: ericstestaccount | Patient ID: [input] | Remaining Assessments: 38 | **Retest** | Logout

5 **ReTest** | Done

Patient ID	Test Time (GMT)
SCNSDJB58	2019-07-09 09:52:56
SCNSOHCTB	2019-07-08 19:25:49

6 Required Identification | Patient ID: SCNSOHCTB | Birth Date: Year: 1955 | Month: Jan | Day: 09 | **OK**

7 Please confirm these settings | Patient ID: SCNSOHCTB | Birth Date: 1955 Jan 09 | Test Language: English (United States) | Tests to perform: [checkboxes for various tests] | **Begin Assessment**