

# CNSVS Web – Administer Neurocognitive Assessments

1


Before you start testing, we recommend checking the following settings:

- Screen Saver is set to "None"
- All power options are set to "Never" or 2 hours minimum.
- Laptop is plugged into power. Running on battery could cause the laptop to shutdown during the test.
- Verify the date is set correctly. Reports are timestamped using the computer's date setting.
- Sticky Keys are disabled on Windows PCs.

2 3 4

- Go to [www.cnsvs.com](http://www.cnsvs.com)
- In the upper right-hand corner of the page select **Sign In**
- Select **Administer Cognitive Tests and Rating Scales**

Sign In

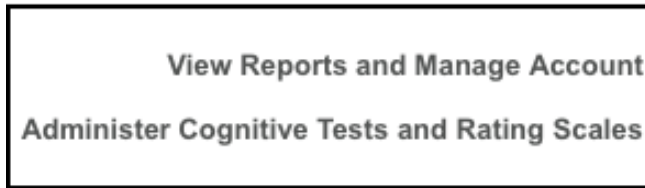

5

- Enter your User Name and Password and then select **Login**



User Name:   
Password:

Login

By continuing, you accept the [CNS Vital Signs Licensing Agreement](#).

6

- Enter a Patient ID and select Test.



Test Administrator: **cnsvs\_support**

Patient ID:  Test

Remaining Assessments: 70

Retest Logout

[Change My Password](#)

**Important:** Create a practice policy for identifying patients. All test results are longitudinally matched based on the unique Patient ID for each patient or research subject.

## Administer Neurocognitive Assessments (web)



# CNSVS Web – Administer Neurocognitive Assessments (page 2)

7

- Enter Date of Birth and select OK.

## Required Identification

Patient ID: 295754  
Birth Date: Year: 1994 Month: Mar Day: 12  
Cancel OK

Date of birth is required to calculate the age matched norms.

8

- Select Test Language, Cognitive Tests, and Scales to perform and then select "Begin Assessment".

Please confirm these settings

Patient ID: 295754  
Birth Date: 1994 Mar 12  
Test Language: English (United States) ← Testing is available in dozens of languages.

Tests to perform:

- Verbal Memory Test
- Visual Memory Test
- Finger Tapping Test
- Symbol Digit Coding
- Stroop Test
- Shifting Attention Test
- Continuous Performance Test
- Perception of Emotions Test
- Reasoning Test
- Four Part Continuous Performance Test
- Adult ADHD Self-Report Scale (ASRS-v1.1) Symptom Checklist LF-18
- Adult Obsessive-Compulsive Disorder Inventory (OCD-A) SF-20
- Alcohol Use Disorders Identification Test (AUDIT) SF-10
- Child Obsessive-Compulsive Disorder Inventory (OCD-C) SF-20
- Depression, Anxiety and Stress Scale (DASS) LF-42
- Depression, Anxiety and Stress Scale (DASS) SF-21
- Dizziness Handicap Inventory (DHI) SF-25
- Drug Use Questionnaire (DAST) SF-20

Neurocognitive Status  
Functional Medicine  
Sleep  
Assessment and Care Planning  
ADHD - Pediatric and Adult  
TBI - Concussion  
Depression - Anxiety  
Memory - MCI  
Pain - Substance Use  
Show All - Clear All

Cancel Begin Assessment

**NOTE:** The patient will be presented with each test selected. When all the test have been completed enter your password to complete the testing process. If "Display report at test completion" is enabled a View Report button will be presented.