

CNS VS Web – View Reports

1 2 3

- ❑ Go to www.cnsvs.com
- ❑ In the upper right-hand corner of the page select **Sign In**
- ❑ Select **View Reports and Manage Account**

Sign In

View Reports and Manage Account
Administer Cognitive Tests and Rating Scales

4

- ❑ Select **View Reports** in the upper left section of your account management page.

- Home
- Place Order/Purchase
- Edit My Account Profile
- View Reports**
- Graph Longitudinal Reports
- Export Data
- Generate Remote Test Code
- View Audit Log
- Manage Users
- Assessment Settings
- Demography Settings

5

- ❑ Select a patient report from the list and then select **View Report** (View Report button is at the bottom of the page, which may be out of view). Double clicking on a patient report in the list will also download the patient report in PDF format.

Select patient record first...

Search Where Patient ID contains Test Dates: Jan 28 2020 To Feb 27 2020

Patient ID	Test Time (local)	User Name	Test Admin	User Group	Completed
295754	2020-02-27 10:25:35	cnsvs_support	cnsvs support		Y
295754	2020-02-19 14:18:47	cnsvs_support	cnsvs support		Y

... then click "View Report"

NOTE: The View Reports page allows you to search your reports by Patient ID and Test Date. The Patient ID search will match all results that contain the characters entered. The Test Date search allows a range of test dates to be searched. The default date ranges is the last 30 days.

